
50A8, 50A9 Omo Immobil Sling

DE	Gebrauchsanweisung	4
EN	Instructions for Use	8
FR	Instructions d'utilisation	12
IT	Istruzioni per l'uso	17
ES	Instrucciones de uso	22
PT	Manual de utilização.....	26
NL	Gebruiksaanwijzing	31
SV	Bruksanvisning	35
DA	Brugsanvisning.....	39
NO	Bruksanvisning	43
PL	Instrukcja użytkowania	47
HU	Használati utasítás	52
HR	Upute za uporabu	56
BS	Uputstvo za upotrebu	60
SL	Navodila za uporabo.....	64
BG	Инструкция за употреба	68
TR	Kullanma talimatı.....	72
EL	Οδηγίες χρήσης	77
RU	Руководство по применению	82
ZH	使用说明书.....	87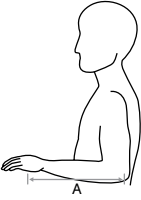

	Größe / Size		A	
			cm	inch
	50A8=S	50A9=S	29-34	11.5-13.0
	50A8=M	50A9=M	35-37	13.0-14.5
	50A8=L	50A9=L	38-41	14.5-16.0

A:

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INFORMATION

Date of the last update: 2014-03-11

- Please read this document carefully.
- Follow the safety instructions.

Explanation of Symbols

CAUTION Warnings regarding possible risks of accident or injury.

NOTICE Warnings regarding possible technical damage.

INFORMATION Additional information on the fitting/use.

1 General Safety Instructions

INFORMATION

The patient has to be instructed regarding proper handling/care of the product.

First fitting and application of the product must be supervised by trained personnel. Daily wearing and application time depends on the medical indication defined by the medical practitioner.

Before use read the safety instructions and directions for use.

CAUTION

The product is intended for use for one patient.

In case of unusual effects (e.g. increasing trouble) take immediate medical advice.

The product must not be changed.

An orthosis/bandage applied too tight may cause local pressure points and possibly constrictions of blood vessels/nerves.

NOTICE

The product is not fire resistant. It must not be worn close to open fire or other heat sources.

The product should not come in contact with aliphatic and acidic agents, salves or lotions.

2 Intended Use

The 50A8 Omo Immobil Sling/50A9 Omo Immobil Sling, abduction shoulder orthosis is to be used exclusively for orthotic treatment of the elbow and is exclusively suitable for contact with healthy skin.

3 Indications and Effects

Indication must be determined by the physician.

50A8 Omo Immobil Sling

All indications requiring immobilization of the shoulder joint in internal rotation, e.g. postoperative and posttraumatic immobilization of the shoulder joint, irritations in the region of the shoulder and shoulder girdle, soft-tissue injuries (e.g. contusions, bruises, distortions), muscle and tendon injuries in the region of shoulder and upper arm, Frozen Shoulder, conservative treatment of AC-joint separation and osteoarthritis, shoulder osteoarthritis.

The shoulder joint orthosis 50A8 Omo Immobil Sling fixes the arm at the body and immobilizes the shoulder joint in internal rotation.

50A9 Omo Immobil Sling, abduction

All indications requiring immobilization of the shoulder joint in 15° abduction, e.g. postoperative and posttraumatic immobilization and relief of the shoulder joint (e.g. after rotator cuff surgery), immobilization after shoulder luxation, soft-tissue injuries (e.g. contusions, bruises, distortions), irritations in the region of shoulder and shoulder girdle (e.g. impingement syndrome, chronic bursitis), shoulder instabilities, follow-up treatment of biceps tendon and muscle injuries.

The shoulder joint orthosis 50A9 Omo Immobil Sling, abduction – consisting of shoulder joint orthosis 50A8 and anatomically shaped waist pad – allows immobilization and relief of the shoulder joint in 15° abduction.

4 Relative Contraindications

In case of the following indications consultation with a physician is required: Skin diseases/-injuries, inflammatory symptoms, prominent scars that are swollen, reddened and feel excessively warm when touched; Lymphatic flow disorders, including unclear soft tissue swelling distant to the body area to which the appliance will be applied; Sensory and circulatory disorders of the arm and hand.

5 Application of Product

The initial fitting and application of the 50A8 Omo Immobil Sling/50A9 Omo Immobil Sling, abduction must only be carried out by qualified personnel. The orthoses are delivered closed and ready for trial fitting. It can be applied on the left as well as on the right side. In delivery condition the orthosis is preset for the right arm.

50A8 Omo Immobil Sling:

Adaptation to the left body side:

Remove the ventral closure attached to the arm loop and fix it on the opposite side (mirror image) in same position.

Application of the orthosis:

- 1) Open the straps. Insert the arm into the loop and adapt the Velcro fastener for upper arm and thumb (fig. 1).

- 2) Place the shoulder strap diagonally over the contralateral (intact) shoulder. Tighten the shoulder strap until the arm is in wished position (fig. 2).
- 3) Close the waist strap (fig. 3). If required the straps can be shortened. Place the shoulder pad in desired position.

3.2 50A9 Omo Immobil Sling, abduction:

Adaptation to the left body side:

Open the closure of the waist strap. Remove the abduction wedge from the arm loop and fix it with the two-sided Velcro fastener on the opposite side (mirror image) (fig. 4). Open the shoulder strap and turn it in closing direction.

Application of the orthosis:

- 1) Open the straps. Insert the arm into the loop. The elbow should be positioned in the closed angle of the arm loop. Place the waist pad on the hip of the affected arm so that the anatomically shaped end points forwards. Adapt the Velcro fasteners for upper elbow and thumb (fig. 5).
- 2) Place the shoulder strap diagonally over the contralateral (intact) shoulder. Close the waist strap (fig. 6). Adjust the length of shoulder and waist strap. If required the straps can be shortened. Place the shoulder pad in desired position.

INFORMATION The position of the shoulder joint orthosis can be adjusted to external and internal rotation by shifting the abduction pad along the hip.

6 Instructions for Care

6.1 Material

The shoulder joint orthosis consists of a breathable, textile-covered foam avoiding skin irritations. Shoulder as well as waist straps are made of velour (polyamide). The abduction pad of Omo Immobil Sling, abduction consists of foam (45kg/m³) and is provided with a Velcro velour cover.

INFORMATION Latex Safe: To our knowledge this product includes no natural rubber latex.

6.2 Cleaning

Washable in lukewarm water with mild detergent. Wash out thoroughly and air-dry.

NOTICE Residues of cleaning agents may cause skin irritations and wear of material.

INFORMATION For washing the Omo Immobil Sling, abduction remove the foam pad from the cover and only wash the cover.

7 Liability

The manufacturer's warranty applies only if the device has been used under the conditions and for the purposes described. The manufacturer recommends that the device be used and maintained according to the instructions for use.

8 CE Conformity

This device meets the requirements of the 93/42/EEC guidelines for medical devices. This device has been classified as a class I device according to the classification criteria outlined in appendix IX of the guidelines. The declaration of conformity was therefore created by Ottobock with sole responsibility according to appendix VII of the guidelines.

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Otto Bock HealthCare GmbH